# Produktkatalog dental

## *Product catalog dental*

**Ihre Kundennummer:**  
*Your customer number:*

**steco-system-technik GmbH & Co. KG**

Kollastr. 6  
22529 Hamburg  
Germany  
[www.steco.de](http://www.steco.de)

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*Steco®, Titanmagnetics®, StecoForm® and logo ® are registered trademarks of steco-system-technik GmbH & Co. KG, Hamburg. There is no need to use the distinctive emblem ® in favor of readability.*

Director Hjalmar Stemmann  
Graduate Economist and  
Certified Dental Technician

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*Dear Customers.*

*Welcome to Steco! We would like to present our company and our products.*

*Our family-run business, steco-system-technik GmbH & Co. KG, was founded in 1996 by my father Hartmut Stemmann and me. However, our dental roots go back over 100 years to when my great-grandfather came back to Hamburg (Germany) as a lay barber after his journeyman years following completion of his barber apprenticeship. In the late nineteenth century prostheses made from the new material, caoutchouc, gave dentistry and maxillofacial prosthetics a new direction. For the first time, it was possible to achieve aesthetically acceptable results with an exogenous substance (not from the body). The development of methyl methacrylate in the 1930s was the next big step. Today, implants and full ceramic restorations are standard practice.*

*The development of application-oriented products has been at the heart of our work since its inventive beginnings. Since the early 1980s, we have been offering magnetic anchoring for our dental implants. We have been working on implant planning since the mid-1990s. Together with users, we have been continuously developing our products. Even tried and tested products can be improved. This means that our products are known worldwide for their outstanding quality and have received multiple awards.*

*We are delighted that you are interested in our products.*

Hjalmar Stemmann  
Geschäftsführer, Inhaber  
Managing Director, Owner

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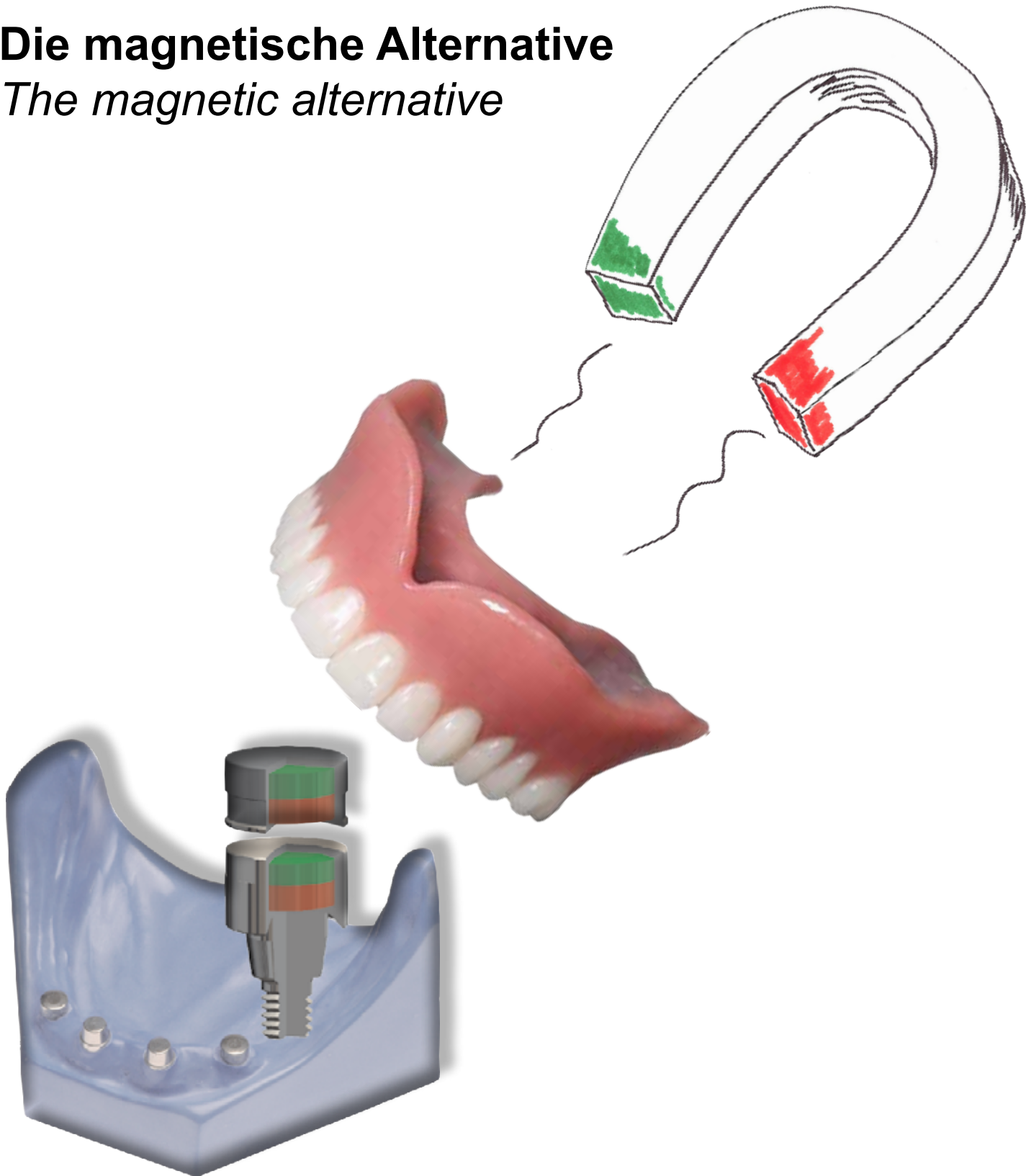
*All marked in the catalog with \*) asterisk product numbers refer to the products that are sold exclusively through selected implant manufacturers. For the correctness of the foreign numbers we take no responsibility. The implant names mentioned on this and the following pages are trademarks of their respective manufacturers.*

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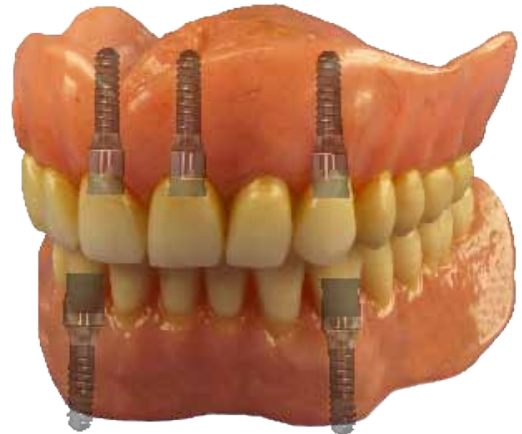
## Magnet attachments for full dentures

This implant-supported solution is for patients aged 80+ with gout or rheumatism, and especially for geriatric dentistry. StecoTitanmagnetics® can also be used with short implants.

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## We have the right attachment for every implant

Can be used with more than 60 different implant connections.

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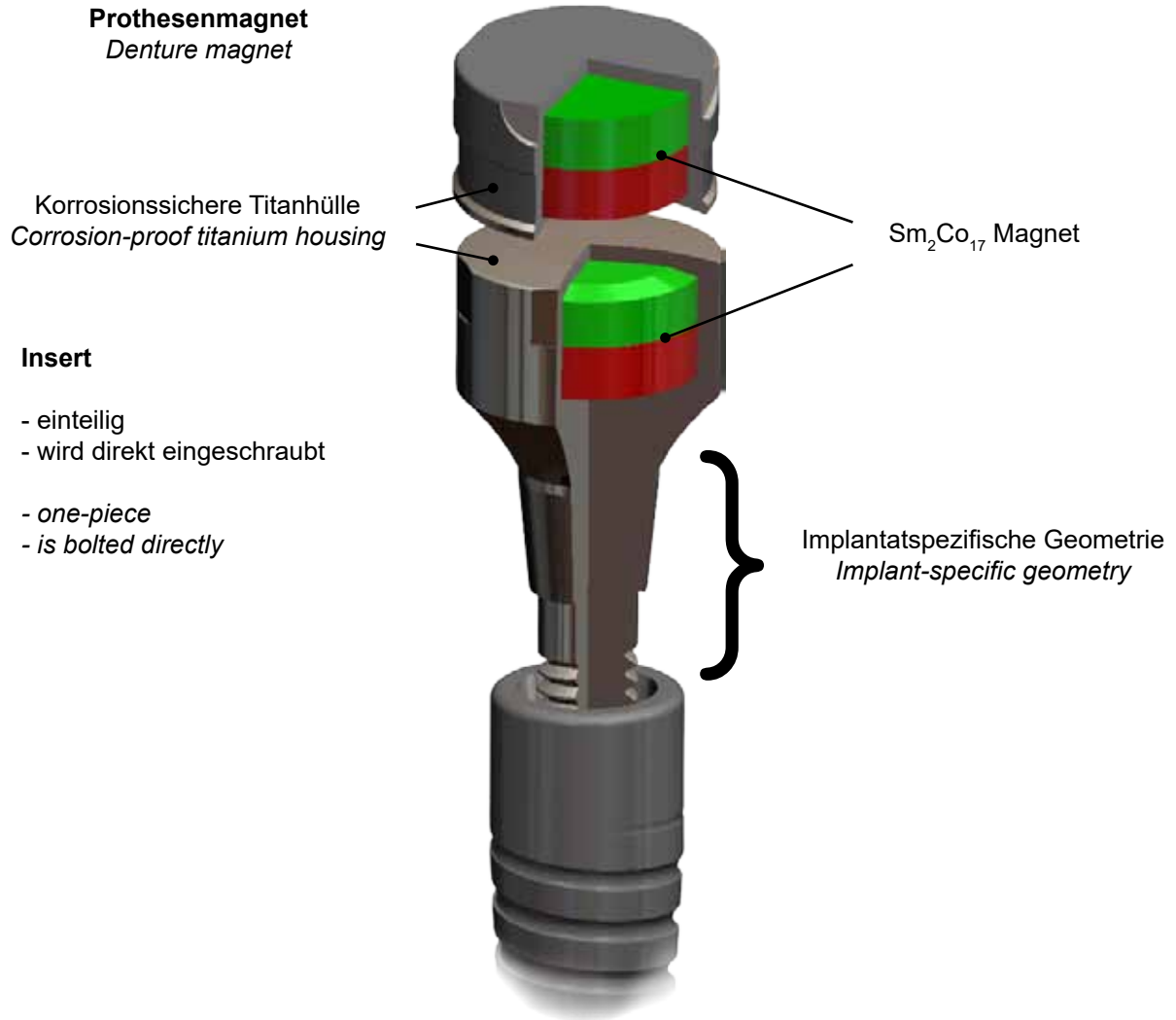
## Advantages:

- easy to use → the denture is easy to put in and take out
- easy to clean → polished surfaces, no undercuts
- value for money → no changing of matrix required
- universal → multiple product lines for all major implant systems

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
### Dual magnet system

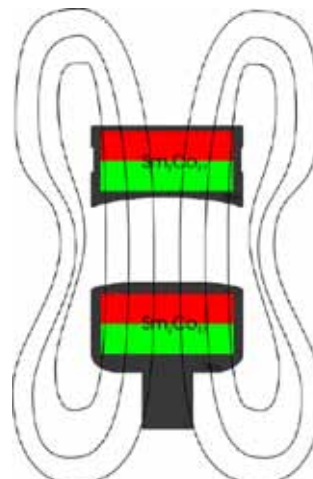Two open magnetic fields exert a force of attraction over a longer distance than mono-magnet systems.

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## Spherical magnets

- slightly curved surface
- self-centering
- exert lowest lateral force into the implant
- especially suited for short implants
- the number at the end of the REF number indicates the height, eg. I.01.02.X325 = 3,25 mm

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## Conical magnets

- conical 10° form
- absorbs lateral forces
- stabilises the denture to prevent sideways movement
- the recommended minimum length of the implant is 12 mm
- the number at the end of the REF number indicates the height, eg. I.01.02.K150 = 1,50 mm

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## Torque wrench insert

- suitable for all common torque wrenches
  - safe handling thanks to coupling
  - safe handling by coupling
- For all compatible ratchets, see page 30.

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## Positioning cuff / resilience ring

The position cuff ensures the denture magnet is polymerised in the right place. The resilience distance prevents the two magnetics coming together.

- protects the implant and tissue
- ideal for chairside use
- resilience distance on the X-Line and Z-Line using the positioning cuff (= 0,3 mm)
- resilience distance on the K-Line using the resilience ring (= 0,3 mm)

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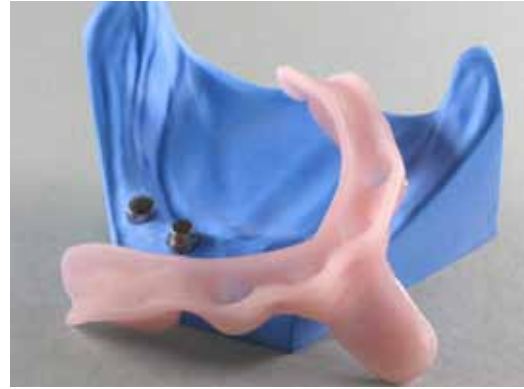
## *Equipment for the laboratory*

*To make a new denture, each product line has its own compatible impression post and laboratory replica.*

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## *Impression post*

- *magnetic impression without screws*
- *lock magnetically to the insert*
- *the external geometry ensures the impression material is securely fixed*
- *for the K-Line, the denture magnet is used*

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## *Laboratory replicas*

- *for making models quickly and hygienically*
- *the original implant does not have to be used on the insert*
- *there is no need for time-consuming cleaning*

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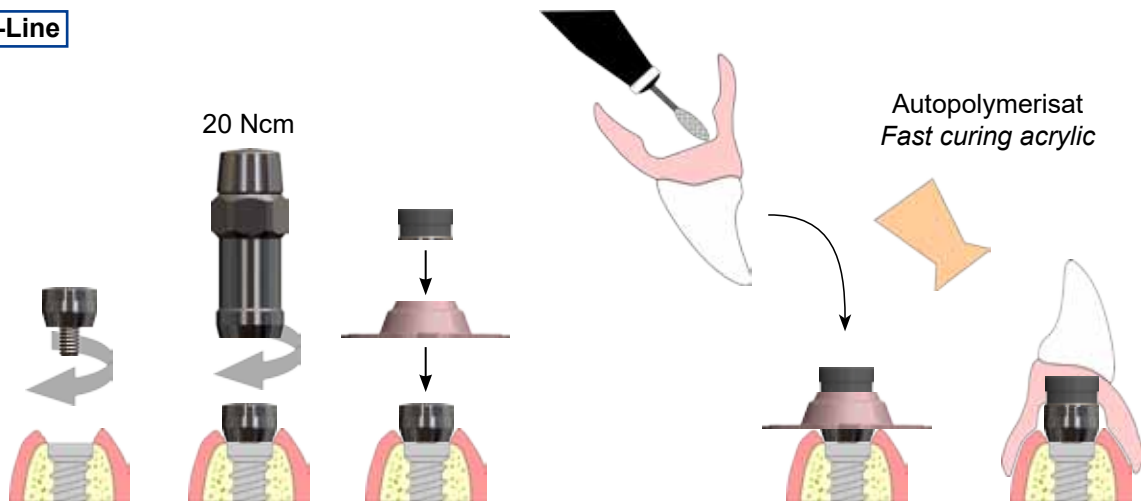
*without magnet,  
magnetic stainless  
steel*

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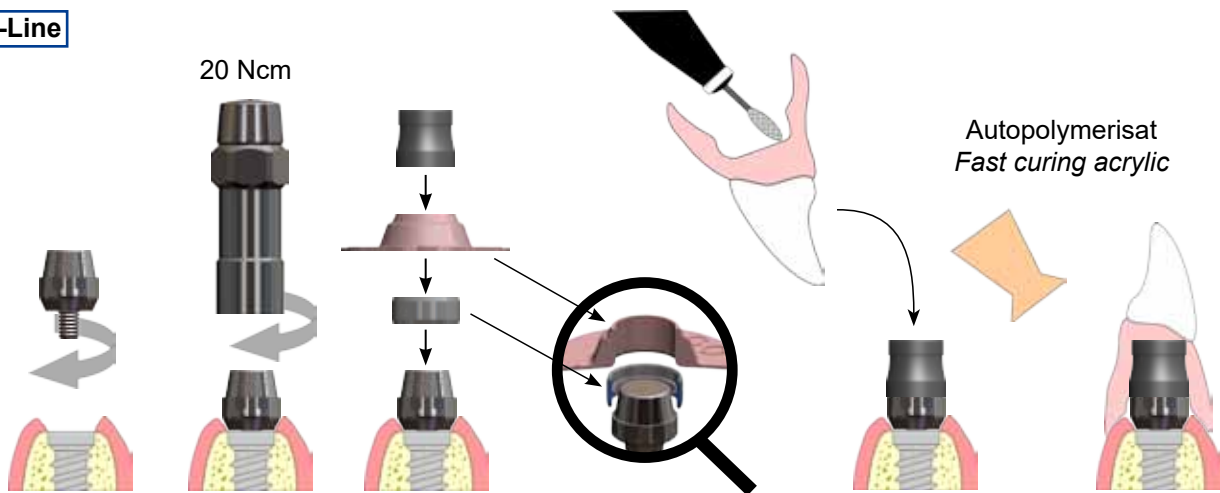
*StecoTitanmagnetics® are screwed with a torque of 20 N/cm. There are torque wrench inserts available for numerous implant-specific torque wrenches (page 28).*

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*Put on the resilience rings, positioning cuffs and denture magnet (for example, on the K-Line).*

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*Fill the polished denture base with curing acrylic, insert the dentures and let the acrylic cure.*

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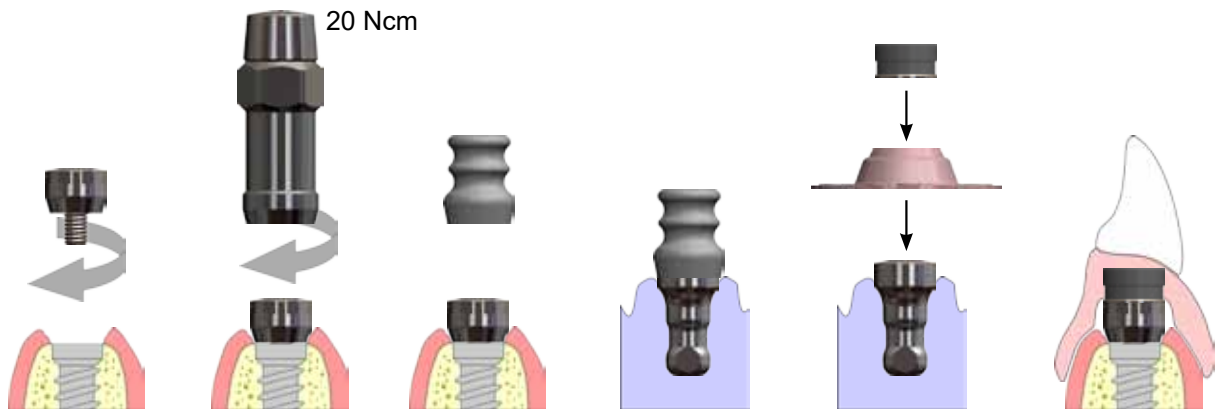
*Basal finishing work on the denture (finished lower denture).*

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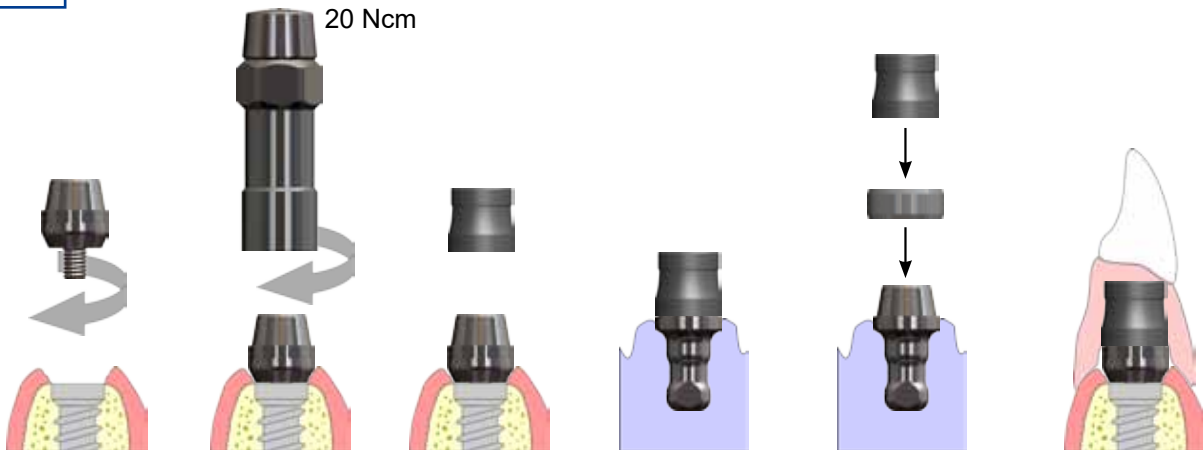
### Drehmomentschlüssel / Torque wrench

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*Demo models are an important tool in patient communication. We offer various demo parts, with which you can make your own models.*

*Our one-piece demo implants combine the functional surface of StecoTitanmagnetics® with an implant. They can be quickly and easily incorporated into a demonstration model.*

Abbildung Figure	Beschreibung Description	REF
	Demoimplantat X-Line inkl. Prothesenmagnet <i>Demo implant X-Line incl. denture magnet</i>	DI.00.35.X1500
	Demoimplantat K-Line inkl. Prothesenmagnet <i>Demo implant K-Line incl. denture magnet</i>	DI.00.35.K1600

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*We can provide you with more printed information for your patients as flyers, as well as our practical patient pass.*

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## Magnets for cast root post caps

Using StecoTitanmagnetics® for root post caps, residual root material can be quickly and easily incorporated into the denture as additional retaining elements.

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## Method

The root post cap is modeled using the usual method, with the help of a castable sleeve and casted into the desired alloy. The corrosion-proof titanium-encapsulated magnet is glued with an orally compatible adhesive into the polished (sand-blasted on the inside) root post cap.

Once the root post cap has been cemented into the patient's mouth, the denture magnet can be glued using the special positioning cuff to the prepared denture right in the dentist's chair. The denture magnets can also be fixed on to the model in the laboratory.

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## Obturator magnets

The root post cap magnets can also be used for coupling obturators. For silicon, the obturator magnets come with a retention ring.

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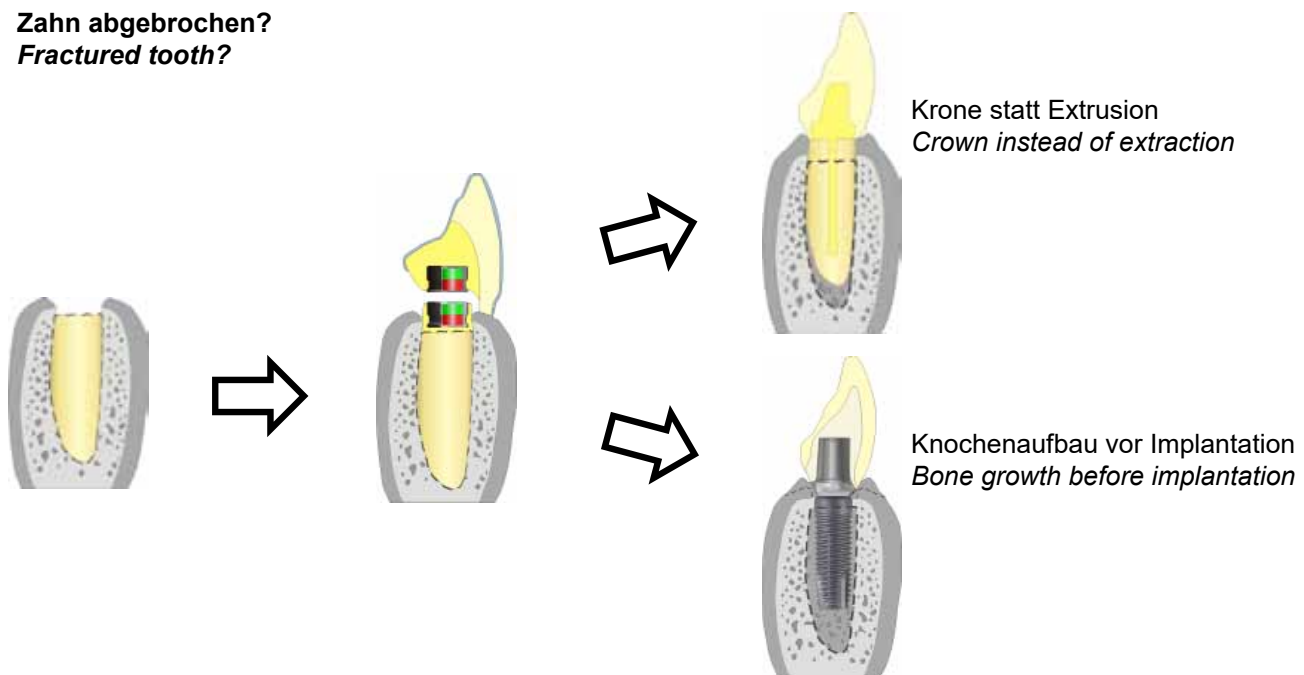
## **Magnetic force moves root and gingiva**

*A corrosion-proof magnet encapsulated in titanium is fixed to the root. A second magnet is fixed to a splint or a temporary denture a small distance away. The magnetic attraction between the two magnets pulls the root out of the alveolus slowly until both magnets meet. Depending on the tooth, the process can take from several weeks up to a few months. The bone tissue below the root is built up. The soft tissue follows the root along the corona.*

*The maximum force between the magnets in contact with each other is about 1 N (100 g). With two positioning aids, the initial distance can be set at 1 mm (0.33 N) or 2 mm (0.13 N).*

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## **Crown instead of extraction**

*Is the root too deeply fractured to attach a crown?*

*With magnetic extrusion, the root can be pulled out of the alveolus so a crown can be placed over it.*

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## **Bone growth before implantation**

*With the help of magnetic extrusion, the patient's own bone and gingival tissue can be pulled upwards. The implant can be inserted more effectively into the body's own bone tissue, ensuring the gums look more aesthetically pleasing.*

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*Unlike magnets available on the Internet, StecoTitanmagnetics® have the CE medicinal product mark.*

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## Indication

**1. Geroprosthetics:** anchoring of hybrid- and partial dentures on class III- (toothless jaw) and class II- (reduced number of teeth) prostheses (classification according to "consensus paper" 12/2008). For class III treatment with removable dentures, 6 implants are recommended in the upper jaw and 4 in the lower jaw. Depending the person's anatomy and the prosthetic requirements, one of the above numbers of posts may be indicated. Due to the negligibly low lateral force on the implant (only applies to X-Line and Z-Line) even short implants (6 mm) can be used.

**2. Maxillofacial prosthetics:** anchoring of facial prostheses and resection prostheses.

**3. Contraindications:** dysfunctions such as bruxism. For StecoTitanmagnetics® K-Line the implants must be at least 12 mm long. StecoTitanmagnetics® T-Line should not be used for oral treatment.

## Warning

When using magnets, special precautions must be taken. When carrying out MRI/MRT scans, implants and dentures must be removed first to avoid damage to the magnets. Keep away from magnetic data storage devices and electronic devices. Pacemakers are not effected by StecoTitanmagnetics® if used correctly, because there is no direct contact (Völkel 1999). Make sure to provide this information to the patient!

Loose inserts may lead to thread breakage and/or damage to the implant thread. Patients should immediately go to their dental practice so that the inserts can be tightened again. It is important to ensure that the prosthesis is funnel-shaped basally around the magnetic head.

The magnet cores can withstand temperatures of up to 250°C/450°F, which means they must not be soldered or welded. The heat would irreversibly damage the magnet. Lasers can perforate the housing. If the titanium housing is damaged, the parts will have to be exchanged as soon as possible. If not, the magnetic alloy ( $Sm_2Co_{17}$ ) would be released exposing it to the oral cavity and causing damage. This can lead to corrosion and a loss of their magnetic force, and eventually to damage of the housing.

Never grind the titanium housing up to 0.2 mm thick!

To manage any risks, damaged parts must be sent back to the manufacturer with product reference and batch numbers, implant date and the implant position. Please record product information (REF, LOT) on the patient file and patient pass!

Some Titanmagnetics products are marked on their label with a UDI code (HIBC) which contains information to the manufacturer (Steco=ESTO) as well as device and production identification.

## Magnetics fields

StecoTitanmagnetics® create a natural and permanent magnetic field like that of the earth. It is not comparable with the electromagnetic field of a mobile phone or high voltage power lines. The average magnetic field on the surface of StecoTitanmagnetics® can reach 186 mT (X and K line) or 300 mT (Z line). This is lower than 40 mT (WHO exposure limit) at a distance of 5 mm from the surface.

There is no evidence in current literature that static magnetic fields near the surface of the magnet with a magnetic flux density of up to 300 milli Tesla can cause local damage in humans. StecoTitanmagnetics® can be damaged by external magnetic fields. Very large magnetic fields, such as those produced by MRT scanners or electric switch gear and control gear can weaken, damage or even reverse the magnetic effect of StecoTitanmagnetics®.

## Sterilisation and disinfection advice

The inserts are not packed in sterile containers. They can be sterilised in their packaging if it is not damaged. We recommend sterilization directly before use. Sterilisation can be performed with moist heat in an autoclave (gravity method or fractioned vacuum 132/134°C, 3 bar, 5 min). Please refer to instructions provided by the sterilisation equipment manufacturer, and only use validated processes. There are separate reprocessing instructions for ratchet inserts.

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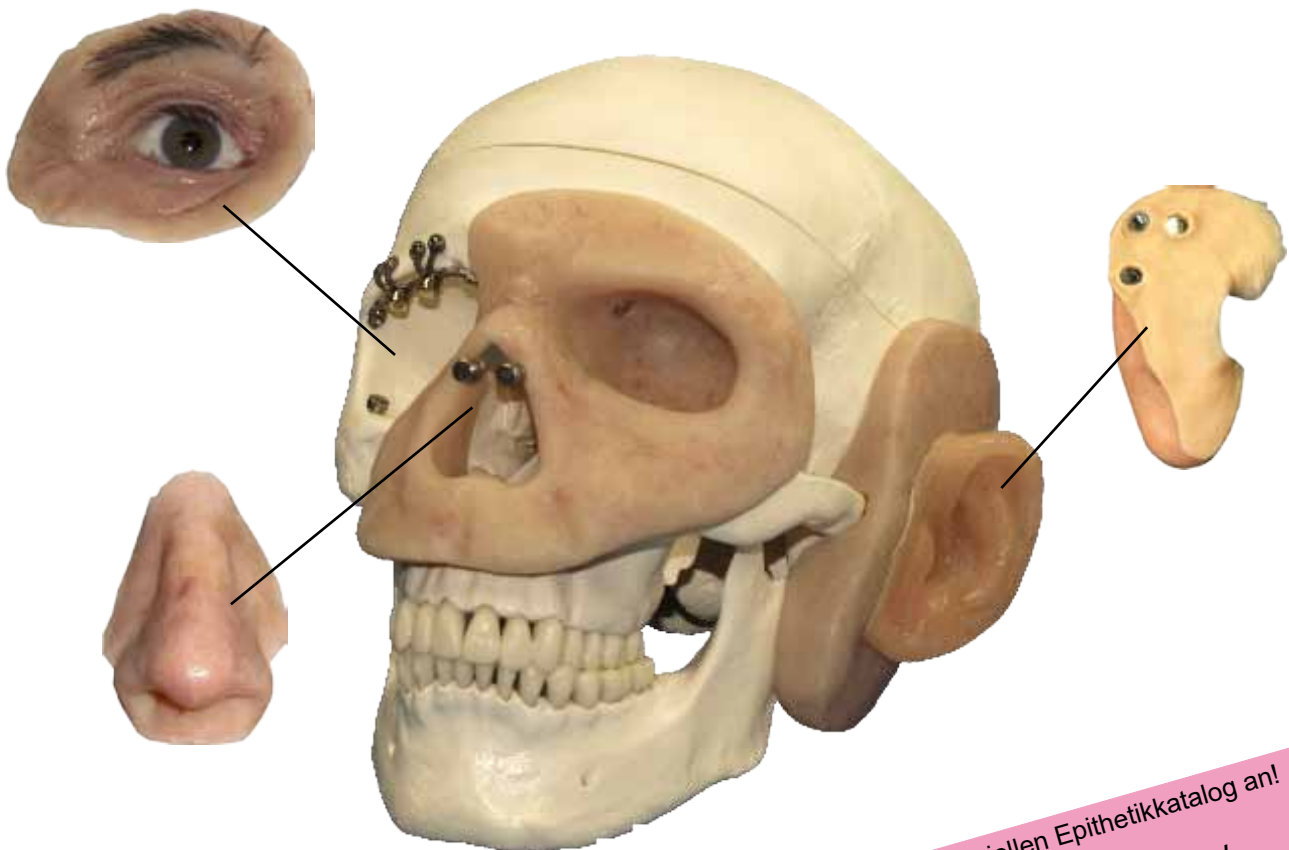
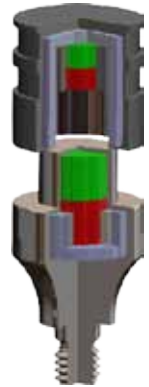
*In the treatment of patients with maxillofacial prostheses, StecoTitanmagnetics® play an important role. We offer an extensive range of StecoTitanmagnetics® specifically for extra-oral use. Special implant and plate systems are used.*

*Our StecoTitanmagnetics® T-Line has been developed exclusively for extra-oral use. Special universal components such as prosthesis magnets with silicone retention or an additional lateral guide have been specially designed for use in maxillofacial prostheses.*

Fordern Sie gerne den speziellen Epithetikcatalog an!  
Ask for the special catalog for facial prosthetics!

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## StecoForm® Flex

*This is an addition vulcanising impression silicone for facial prosthetics. With its low hardness and thixotropic behaviour, StecoForm® Flex is especially suited for creating impressions of undercuts in facial and corrective prosthetics.*

*StecoForm® Flex (medium viscous - type 2) is identical to the well-known and popular impression material, Epiform Flex, by Dreve Dentamid. It comes in the well-known 50 ml double cartridges.*

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### Technical data:

- final hardness: 12 Shore A
- processing time: 1:30 min ± 15 s
- deflasking time at 37° C: 3 min ± 1 min
- Linear change of measures: < 0.1 %
- Deformation under pressure: 16.1 ± 0.5 %
- Recovery from deformation: > 95 %
- Color: orange

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*From simple diagnostic planning to a full digital implant process chain, a range of different sleeves can be used. StecoGuide offers different types of drill sleeves for planning and drilling templates.*

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### **Which drill sleeve for what?**

*The drill sleeve to be chosen depends on the planned type of drilling template. For simple planning templates, different sleeves are required from those used for surgical templates, with which guide drills and the implant have to be guided. In addition, drilling accuracy requirements will determine choice of sleeve diameter.*

- *planning*
- *pilot drilling*
- *multiple drilling stages*
- *fully guided*

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
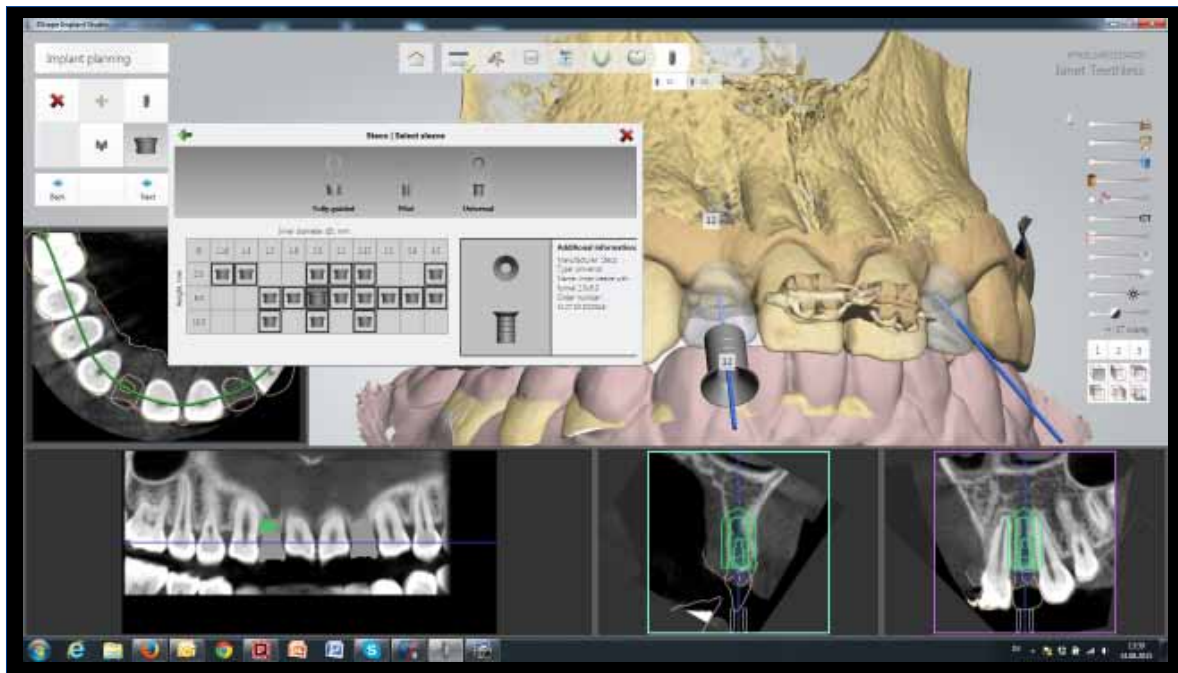
## Which sleeves for which software?

The geometry data of Steco sleeves are stored in many implant planning programs. In some programs, the sleeves have different names. Example: for 3Shape Implant Studio, the titanium single sleeve is called a pilot sleeve and the titanium double sleeves are called universal sleeves.

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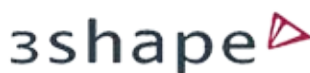*For radiographic diagnosis, the titanium reference balls are suitable. They help to determine whether the planned implant position is surgically feasible. There are less artifacts in the X-ray image than steel balls.*

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## *Titanium reference balls*

- ø 5.00 mm - simple planning tool, eg. for mucosal thickness
- ø 2.50 mm - position marker

REF	D [mm]	VE
M.27.09.D500	5,00	10
M.27.09.D250	2,50	3

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*For radiographic diagnosis, the titanium single sleeves are suitable. They help to determine whether the planned implant position is surgically feasible. They can be used for pilot drill as well.*

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### *Titanium single sleeves*

- particularly well suitable for use in planning templates
- easy to measure in X-ray images (titanium)
- universal diameter (ø2.35 mm standard drill shank)
- simple surgical guide

REF	D [mm]	d [mm]	L [mm]	VE
M.27.01.D200L5	3,00	2,00	5,00	10
M.27.01.D235	3,00	2,35	10,00	10
M.27.01.D235L5	3,00	2,35	5,00	10

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*The drill produces a hole that is compatible with the sleeve diameter into which the sleeve has to be pressed. The impression tool makes it easier to push in the sleeve.*

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## Titanium outer sleeves

In the outer sleeves (inner  $\varnothing$  3.50 mm), the inner sleeves (outer  $\varnothing$  3.50 mm) can be inserted into each other accurately. With this **double sleeve principle** (tube-in-tube) the outer sleeves sits securely in the template. The inner sleeves can be changed with the different drill diameters. The outer sleeves can be used individually, as well as the inner sleeves.

REF	D [mm]	d [mm]	L [mm]	VE
M.27.02.D350	4,00	3,50	6,00	10
M.27.02.D350L5	4,00	3,50	5,00	10

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## Open outer sleeves

This special outer sleeve is suitable for limited spaces because the drill swiveled over entire sleeve length. Swivel the inner sleeve upwards, guide it down and prevent it from tilting.

REF	D [mm]	d [mm]	L [mm]	VE
M.27.18.D350	5,00	3,50	6,00	5

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## Equipment for outer sleeve

The template drill produces a hole in the template that fits the external diameter. Than the sleeve just has to be pressed in (e.g. with the impression tool). The sleeve holder is used to position the outer sleeve when polymerising or gluing.

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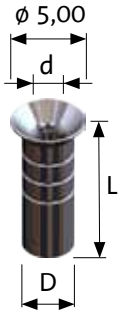
### Titanium inner sleeves

They fit exactly into the outer sleeves, but can also be fixed directly to the template. They are offered with many different drill diameters (inner- $\emptyset$ ) - from 1.50 mm to 2.80 mm.


REF	D [mm]	d [mm]	L [mm]	VE
M.27.03.D150L6	3,50	1,50	6,00	10
M.27.03.D150	3,50	1,50	10,00	10
M.27.03.D160L6	3,50	1,60	6,00	10
M.27.03.D200	3,50	2,00	10,00	10
M.27.03.D200L6	3,50	2,00	6,00	10
M.27.03.D220L6	3,50	2,20	6,00	10
M.27.03.D235	3,50	2,35	10,00	10
M.27.03.D235L6	3,50	2,35	6,00	10
M.27.03.D250L6	3,50	2,50	6,00	10
M.27.03.D280L6	3,50	2,80	6,00	10

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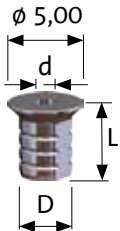### Titanium inner sleeves with depth stop

They are suitable for drills with small depth stop and they are many different diameters - from 1.16 mm to 2.35 mm.

REF	D [mm]	d [mm]	L [mm]	VE
M.27.24.D116L5	3,50	1,16	5,00	10
M.27.24.D130L5	3,50	1,30	5,00	10
M.27.24.D200L5	3,50	2,00	5,00	10
M.27.24.D220L5	3,50	2,20	5,00	10
M.27.24.D235L5	3,50	2,35	5,00	10

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### Equipment for inner sleeves

The template drill is used when the inner sleeve is to be pressed directly in the template.

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*For fully guided surgery special surgical instruments are used, which are guided in guide sleeves (or master sleeves). The instruments have cylindrical guide surfaces and a depth stop or are guided in suitable inserts (tray/keys)*

- for "full-guided" surgical kits
- diameter and length adjusted to the guide sleeves of established surgical kits
- alternative sleeves for open planning systems
- uses less space than circular sleeves

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*Anchor pins are used for stabilising drilling templates in jaws with no or few teeth. Several anchor pins are placed in the template through corresponding sleeves.*

- for 1.5 mm anchor pin drills and anchor pins
- for stabilisation of drilling templates

REF	D [mm]	d [mm]	L [mm]	VE
M.27.20.D150L10	3,50	1,50	10,00	10

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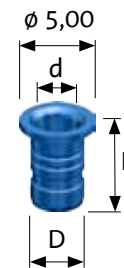
*The double sleeve system was developed especially for the drill instruments of Thommen Medical. Tube-in-tube-principle. Not compatible with the universal titanium double sleeves!*

- inner sleeves for VECTODrill™ pilot drill  $\varnothing$  2.0 and twist drill  $\varnothing$  2.8 mm
- outer sleeves for VECTODrill™ twist drill  $\varnothing$  3.5 mm


## Thommen Medical Titan-Innenhülsen / Titanium inner sleeves

REF	D [mm]	d [mm]	L [mm]	VE
M.27.25.D202L6	3,55	2,02	6,00	10
M.27.25.D288L6	3,55	2,88	6,00	10

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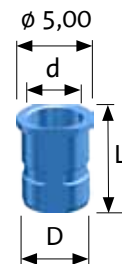
**A preparation of obliterated root canals can be difficult because it cannot be controlled.**

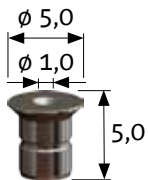*With a 3D planning software the drilling canal is defined (e.g. with coDiagnostiX™ or other systems). Drill- and sleeve geometry can be integrated in the software and can be inserted into a milled or printed surgical template. The ATEC twist drill is precisely guided by the StecoGuide Endo-Sleeve.*


**StecoGuide Endo-Hülse für ATEC Bohrer / StecoGuide Endo-Sleeve for ATEC twist drill**

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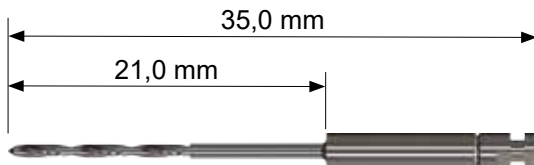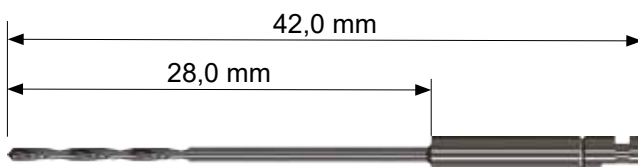
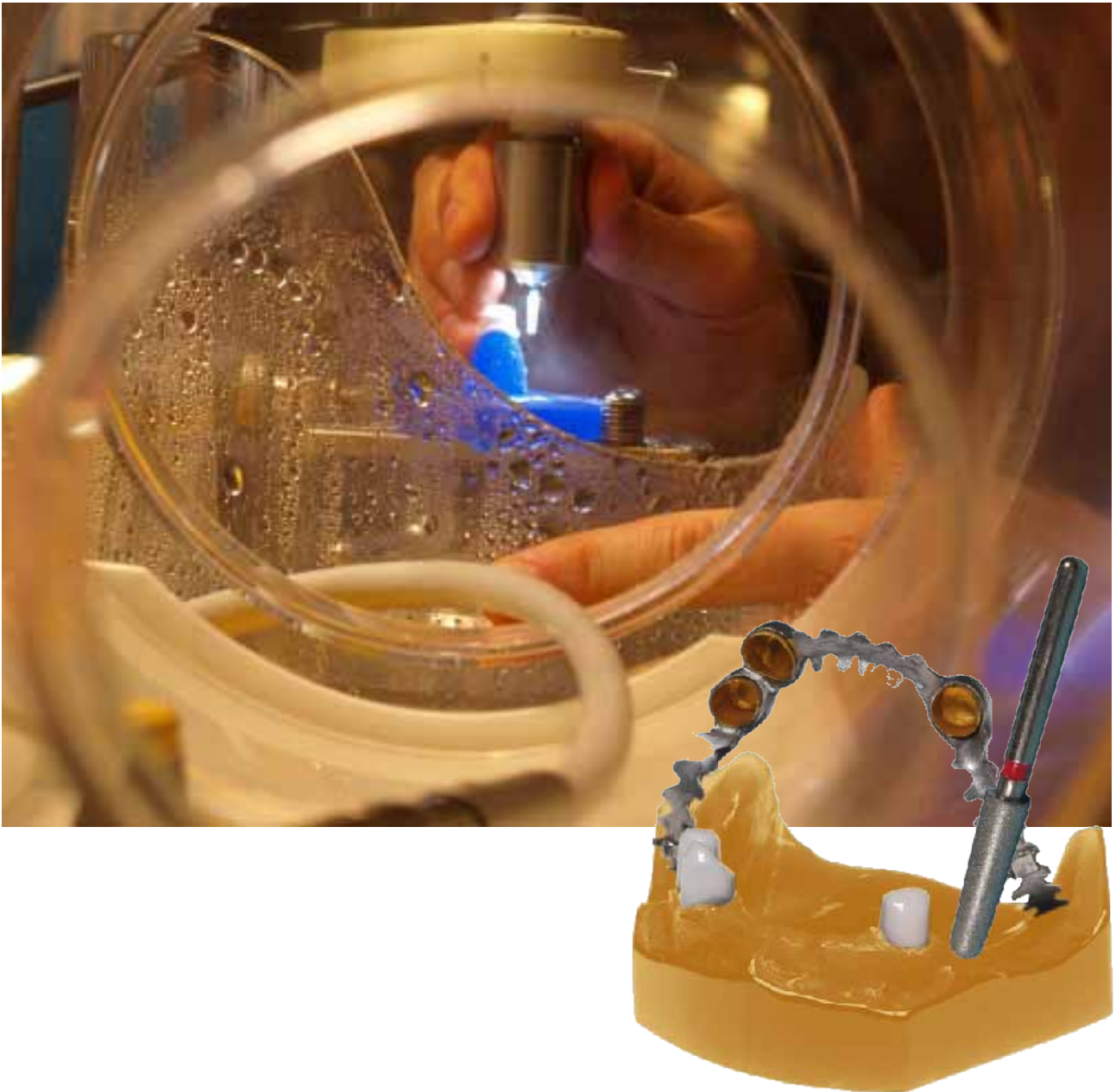
### System expansion:

*Further drilling steps through StecoGuide outer and inner sleeves in different diameters are possible. With the tube-in-tube principle, several drills with different diameters can be guided in a template. We offer numerous sleeves with diameters from 1.16 mm to 3.50 mm. The endo-sleeve is compatible with our double sleeve system.*

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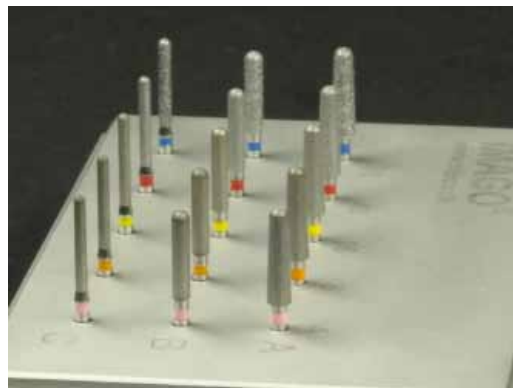
## Grinders for ZrO<sub>2</sub> attachments and telescopes

*Conical or telescopic crowns and bridges made of zirconium dioxide or other high-performance ceramics need perfectly polished surfaces. Diamond grinders can be used to polish zirconium dioxide with a water-cooled turbine (without additional pastes). StecoGrind diamond grinders produce a polished surface in five increasingly finer steps (80, 40, 15, 8, 4 µm). The finest level has diamonds 4 µm in size, creating a glaze-like surface. No dry polishing is needed.*

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*The grinders fit into any turbine handpiece with FG fitting (1,6 mm) and are available in 2 ° and 0 ° (ø 1.2 and 1.7 mm).*

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*The set includes all three figures (2° + 0° thick und 0° thin) in all 5 grain sizes (80 µm bis 4 µm).*

<b>REF</b>
S.44.01.S15

## Grind Schleifkörper / Diamond tools

Einzelbestellung / Single order

Abbildung Figure	Figur Shape	Körnung Grain size	REF	Farbe Colour	Empfohlene Drehzahl Recommended speed
	A	80 µm	O.44.01.A80		300.000 rpm
	A	40 µm	O.44.01.A40		250.000 rpm
	A	15 µm	O.44.01.A15		200.000 rpm
	A	08 µm	O.44.01.A08		150.000 rpm
	A	04 µm	O.44.01.A04		120.000 rpm
	B	80 µm	O.44.01.B80		300.000 rpm
	B	40 µm	O.44.01.B40		250.000 rpm
	B	15 µm	O.44.01.B15		200.000 rpm
	B	08 µm	O.44.01.B08		150.000 rpm
	B	04 µm	O.44.01.B04		120.000 rpm
	C	80 µm	O.44.01.C80		300.000 rpm
	C	40 µm	O.44.01.C40		250.000 rpm
	C	15 µm	O.44.01.C15		200.000 rpm
	C	08 µm	O.44.01.C08		150.000 rpm
	C	04 µm	O.44.01.C04		120.000 rpm

## Zubehör C.M. Turbine / Equipment C.M. turbine

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The set includes 18 diamond tools in different shapes and grain sizes. They have been developed for post-processing of zirconia in the dental laboratory.

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*The tube can be used for freehand working with the turbine. It is rotatably supported and can be used even under the microscope. The waste water drains through a hose into a separate canister.*

- protects the workplace from water
- for hands-free grinding
- usable under a microscope
- for left and right-handed users
- includes drain hose

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*The pot has a metal base and can be mounted securely to almost any milling device with magnets, to produce ceramic double crowns, milled bars and attachments with the turbine. The waste water drains through a hose into a separate collecting vessel.*

- protects the milling device when working with the turbine
- fits all milling machines
- can be attached to a magnetic plate with a metal base
- includes drain hose

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*Please call us to find out about our other Equipment: our silver varnish airbrush (Argentobrush) and our opaque airbrush (Layerbrush)!*

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## **Safety, liability and warranty**

Advice is given verbally, in writing, via electronic media or through demonstrations by our trained medical device consultant. Nevertheless, this does not exempt the user from the obligation of personally testing the product to ensure that it is suitable for the intended purposes/procedures. The user is also obliged to keep up to date with Steco® products (e.g. via visiting our website at [www.steco.de](http://www.steco.de)) and find out information relating to product application. This applies especially for techniques that have not been expressly recommended. In cases of doubt, users should contact steco-system-technik GmbH & Co. KG.

The user is solely responsible for product processing and application as this is beyond our control. We assume no liability for any damage arising during processing and application. In the case of intra-oral application, our products must generally be safeguarded against aspiration. We guarantee the perfect quality of our products within the framework of our general terms and conditions. Information relating to the existence of patents, trade names or other intangible asset legislation are legally non-binding.

Steco® products must be used only with original components and instruments supplied by the steco-system-technik GmbH & Co. KG. Please comply with the instructions for use and the product data sheet. We assume no liability for warranty or product performance if other components manufactured by a third party are used, as product function can be compromised. The technical advice supplied in relation to the use of our products is based on state-of-the-art science and technology valid at the time of product launch.

## **Please note**

We urgently recommend that you receive instruction in the handling of our products from an experienced user. Steco® products should only be used by doctors, dentists, surgeons, dental technicians and anaplastologists who are familiar with the system.

## **Validity**

The publication of the 09.2018 catalogue invalidates all previous information.

## **Sale, availability**

Our products are sold only to dentists, doctors, surgeons, clinicians, dental laboratories and anaplastologists or to persons acting on their behalf. Not all components are universally available. Other trademarks, especially in conjunction with the names of implants, are used regardless of their unconditional availability. The specified product names are the trademarks of the respective manufacturers. Titanmagnetics®, Steco®, coOrdination® und IMAGO® are listed trademarks of steco-system-technik GmbH & Co. KG.

## **Packaging unit**

Unless specified otherwise, each pack contains 1 unit.

## **Copyright**

Re-print or publication, also in extracts, is permitted only with the written approval of steco-system-technik GmbH & Co. KG, 22529 Hamburg, Germany and specimen copy.

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*monday to friday  
8:30 a.m. to 5:00 p.m.*

*Telephone: +49 40 55 77 81-0  
Skype: steco.beratung2*

*Out of our business times you can give your order by mailbox/voice,  
by telefax +49 40 55 77 81-99  
or by e-mail [info@steco.de](mailto:info@steco.de)!*

*Our distribution is responsible for Germany and all the  
other countries where we have no distribution partner.*

*Our distribution partner:*

*Switzerland: Kaladent AG, Urdorf*

*Spain: KUSS Dental, Madrid*

*Poland: HT Dental, Germany*

*Great Britain: techceram, West Yorkshire*

*France (only sleeves): Bio Summer 3D, Paris*

*USA (only sleeves): Aeton Medical, Philadelphia*

*Australia (only sleeves): Dental Axess*

*Italia (only sleeves): CMF Marelli*

***We have a lot more partners all over the world.  
Please check our website!***

***You decide, how fast we deliver!***

*Incoming order for shipment at the same day  
monday to friday till 3:00 p.m.*

*Cost of shipment in other countries than  
Germany are different. Please ask us!*

- during 14 days*
- Pre-Announcement by telephone or e-mail*
- only in undamaged packaging*
- please fill in the slip of return*
- please take absolute a padded envelope or a box/carton*
- no acceptance of unfree shipments*
- in case of losing products due to uncorrect packaging,  
we cannot give a credit*

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*As a responsible manufacturer of medical devices, we obviously use a quality management system.*

*"Quality is the fulfillment of requirements" – this is our motto for developing, designing, producing and distributing our products. Our quality management system is EN ISO13485: 9001 2003 + AC 2009 and EN ISO 9001:2008 certified.*

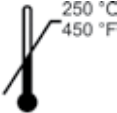
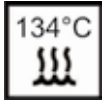
*We are very keen to receive feedback from our customers. Only through the information provided by our users, can we make our products even better. If you have any complaints about our products, please let us know. We welcome your suggestions.*

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# General terms and conditions

## 1. General, Scope of Application

(1) These General Terms and Conditions (hereinafter called the GTC) apply to all business relationships within the steco-system-technik GmbH & Co. KG. The steco-system-technik GmbH & Co. KG maintains a permanent branch at 32657 Lemgo, Alkenbrede 3. These Terms and Conditions apply if the customer is an entrepreneur (§ 14 of the German Civil Code), a legal entity under public law or a special fund under public law. A conclusion of contract is not made with consumers (§ 13 of the German Civil Code).

(2) The GTC apply notably for contracts concerning the sale of fine-precision mechanical products and medical products, in particular for implant dentistry and modern dental technology. Unless otherwise stated, the GTC shall apply in the version valid at the time of the conclusion of contract. In case of deviations between the former named version and the last text version of which the customer is informed, the latter is to be adhered to. This contract serves as a framework agreement which also applies to similar contracts, without us having to refer to these GTC again for every individual case.

(3) Our GTC shall apply exclusively. Deviating, conflicting or supplementary General Terms and Conditions of the customer shall only become part of the contract once we have explicitly approved their validity. This approval requirement applies in every case, for example also if, with the knowledge of the General Business Terms of the Buyer, we carry out the delivery.

(4) Individual agreements reached with the customer in individual cases (including side agreements, additions and amendments) shall have priority over these GTC. The content of such agreements shall be, subject to evidence to the contrary, governed by contract or our written confirmation.

## 2. Conclusion of contract

Our offers are noncommittal. The ordering of goods by the customer is valid as a binding offer of contract. The acceptance of the contractual offer by steco-system-technik GmbH & Co. KG can be concluded either in writing or by delivery of the goods. Orders for special manufacturing by § 3.8 of German Medical Device Act require the written form with the necessary statements.

## 3. Right of modification

We reserve the right to make changes to the materials used or to the construction which deviate from the product description in publications (e.g.: catalogues and technical documentation), provided that they do not substantially impinge upon the normal use or upon the contractually agreed utility of the goods and as long as warranted characteristics are not concerned.

## 4. Delivery times

(1) The delivery time shall be agreed on individually or indicated by us when accepting the order. If this does not happen, the delivery period is three weeks from the date of the contract.

(2) Insofar as we cannot observe binding delivery deadlines for reasons for which we are not responsible, (non-availability of the service), we shall notify the customer immediately and inform them of the estimated new delivery. When the goods or services are not available by the new delivery date, we shall be entitled to rescind the contract in whole or in part; we will immediately reimburse any remuneration that has been provided by the buyer. A case of non-availability of the service is in particular the late self-delivery by our components suppliers if we have concluded a congruent hedging transaction and neither we nor our supplier was at fault or we are not obliged to the procurement in a particular case.

## 5. Delivery, risk assumption, purchase and default of acceptance

(1) The delivery is carried out from the office of the steco-system-technik GmbH & Co. KG, which is also the place of performance and the place of any supplementary performance. If the product was ordered in the permanent branch Lemgo, then this will be the place of performance too. At the request and expenses of the customer, the goods will be delivered to another place of destination (sending purchase). Unless otherwise agreed, we are entitled to determine the type of shipment (in particular transport company, shipment route, and packaging) and to undertake partial deliveries and performances, provided this is reasonable for the customer.

(2) The risk of accidental destruction or accidental deterioration of the goods passes eventually over to the customer with transfer of the goods. In the case of shipment purchases, the risk of accidental destruction or deterioration of the merchandise is transferred to the customer with the transfer of the merchandise to the forwarding agent or other parties or agencies authorized for shipment. If an acceptance procedure has been agreed on, this is authoritative for the passing of risk. The statutory regulations of the works contract also apply to the agreed acceptance in accordance with the law. It is deemed equivalent to the handover or acceptance if the buyer is in default with the acceptance.

(3) In case the buyer defaults the acceptance, fails to act in cooperation or our delivery is delayed for any other reason, we are entitled to demand a claim for compensation for the damage resulting from it, including additional expenses (such as costs for warehousing). For this we will charge a lump sum compensation amounting to 0.5 % of the value of the stored merchandise per commenced month, as long as it is not proven that the expense was lower. The right to assert further proven costs is reserved. The proof of higher damages and our statutory claims (in particular reimbursement of additional expenses, reasonable compensation, termination) remain unaffected. The flat rate is however to be offset against further monetary claims. The customer is at liberty to provide evidence that we have incurred no damages, or that damages were considerably lower than the flat-rate amount stated above.

## 6. Prices and Terms of Payment

(1) Insofar as not otherwise agreed in an individual case, our actual prices which respectively apply at the time of the conclusion shall apply from the place of performance (permanent establishment or Lemgo), plus the legal value added tax. Our prices are listed in Euros, and exclude packaging, freight and insurance.

(2) The customer pays for the cost of carriage from the registered office of steco-system-technik GmbH & Co. KG or branch Lemgo and the cost of insurance chosen by the customer, if the goods are shipped at the request of the customer (5.2 GTC). Any customs duties, fees, taxes and other public charges are to the customer's account.

(3) Invoices are payable in cash or by advanced bank transfer to the account specified in the bill. All invoices shall be due and payable, strictly net cash and immediately upon receipt without any deductions or charges to steco-system-technik GmbH & Co. KG. All payments from foreign countries shall be made in Euros. Cheques will only be available for processing until they have been cashed. The maker of the cheque shall bear the costs of the submission. If a direct debit has been arranged (SEPA), the customer will be informed 2 days before the amount will be charged. Our Creditor Identifier is: DE29ZZZ00000500743. Within the scope of current business relations, we shall be entitled to demand either a complete or a partial prepayment for the delivery. We will endeavour to stipulate a reservation with confirmation of the order at the latest.

(4) All options for payment, which are stated in the online shop of steco-system-technik GmbH & Co. KG during the order process, are at the customer's disposal. Basically, the customer can pay by credit card, PayPal (<https://www.paypal.com/webapps/mpp/home>) or by direct debit based on SEPA.

(5) Where the payment term is exceeded, the customer shall be in default of payment. Interest is to be paid on the purchase price at the respective applicable interest rate for default during the default. We reserve the right to claim any further damages from default. Our claim for the commercial maturity interest (§ 353 of the German Commercial Code) against merchants remains unaffected.

(6) The customer is only entitled to set-off rights and rights of retention if his counterclaims have been established as final and absolute or they are undisputed Reciprocal rights, which the buyer is entitled to claim against the seller in the event of defects in the delivery remain, in particular of part 8.4 this GTC, untouched.

(7) If there are indications after the conclusion of the contract that our entitlement to the purchase price is at risk through insufficient ability of the buyer to pay (e.g. by an application for opening of insolvency proceedings) then according to the statutory regulations we are entitled to refuse service and - if applicable after setting a deadline - to cancel the contract (§ 321 of the German Civil Code).

## 7. Reservation of title

(1) We reserve the right to the property of the sold goods until the full payment of all of our current and future claims from the purchase contract and a current business relationship (secured claims).

(2) Prior to the complete payment of the secured claims, the goods for which ownership is reserved can neither be pledged nor assigned by way of security to third parties. The customer is required to inform us immediately, if an application is made for opening insolvency proceedings or if third parties gain possession of the goods belonging to us (e.g. attachments).

(3) Where the client acts contrary to the contract, in particular if claims which are due for payment are not paid as per the contract, we are entitled to cancel the contract according to the statutory regulations and to request that the goods are handed over owing to the reservation of title. Any demand for the return of goods shall not be deemed to include a simultaneous declaration of withdrawal; instead we shall be entitled to only claim return of the goods and reserve the right of withdrawal. If the customer does not pay the purchase price due, we may assert these rights only if we have first set the customer an appropriate time limit for payment without result or if setting a time limit may be dispensed with according to the provisions of law.

(4) The customer is entitled to sell or process the goods under retention of title in the ordinary course of business. In this case the following provisions

shall apply in addition:

- (a) Reservation of ownership shall also apply to goods resulting from the processing, mixing or combining of our goods in their full amount, whereas these processes are performed on our part so that we are considered as manufacturer. If the ownership rights of third parties remain in existence during the processing, mixing or combination with their goods, we shall acquire co-ownership in relation to the objective value of these goods. Incidentally the same shall apply to the produced product as to the goods delivered under reservation of title.
- (b) The buyer hereby now already assigns the claims against third parties, which are established from the resale of the goods or production total or in the amount of our possible co-ownership share, to us as collateral according to the aforementioned paragraph. We herewith accept such assignment. The customer's duties under number 7.2 hereof also apply to the claims assigned.
- (c) Beside us the customer remains authorized to collect the claims. We engage to leave the claim uncollected for as long as the customer fulfils their obligations of payment towards us, there is no lack of his financial capacity occurs and as long as we do not claim the reservation of title by any other rights under number 7.3. of these GTC. Should this be the case, however, we may demand that the customer discloses to us the assigned claims and their debtors, that he provides all necessary information and surrenders all appropriate documents and that he notifies the debtors (third parties) of the assignment. Furthermore, we have the right to revoke the customer's authority for resale and the right of processing for the goods subject to retention of title.
- (d) If the realizable value of the securities granted exceeds our claims against the customer by more than 20%, we shall be ready at the customer's request to release securities of our choice.

## 8. Warranty claims of the customer

- (1) The rights of the customer concerning material defects and defects of title (including incorrect and short deliveries as well as improper installation or inadequate operating installation instructions) are subject to the legal regulations as long as no alternative is defined hereinafter. The special provisions on final delivery to a consumer (supplier recourse pursuant to §§ 478, 479 German Civil Code) shall remain unaffected at any time.
- (2) The primary basis of our liability for defects shall be the agreement made concerning the quality of the goods. All product descriptions serve as an agreement about the quality structure of the products which are the object of the individual contracts, while there is no difference whether the respective product description originates from the customer or from us.
- (3) The validity of any claims on the part of the customer shall be contingent on the customer having duly fulfilled his obligation to inspect the merchandise and to notify us of any defects, as required by § 377 of the German Commercial Code. If a defect is determined during the inspection or subsequently then this is to be reported to us immediately in writing. At the same time, the same shall be reported in respect of the carrier. The report is deemed as immediate if it is made within two weeks whereby the timely despatch of the report is sufficient in order to safeguard the deadline. Irrespective of this obligation for inspection and reporting of complaints the customer must report obvious defects (including false and shortfall in delivery) within two weeks from delivery in writing whereby the timely despatch of the report is also sufficient here in order to safeguard the deadline. If the customer fails to carry out the proper inspection and/or report defects our liability for the defect which was not reported is excluded.
- (4) If the delivered object is faulty we can initially choose whether to provide subsequent performance by remedying the defect (subsequent improvement) or by delivering a faultless object (substitute delivery). Our right to refuse the chosen type of subsequent performance under the statutory prerequisites remains unaffected. We are entitled to make the owed subsequent performance dependent on the fact that the customer pays the due purchase price. The customer is however entitled to retain a part of the purchase price which is proportional to the size of the defect.
- (5) The customer has the duty to give us the time and occasion for the subsequent costs owed, especially to hand over the faulty goods for inspection reasons. In the event of the substitute delivery the customer must return the faulty object to us according to the statutory regulations. The supplementary performance does neither constitute the disassembling of the defective goods nor the reinstallation, if we initially were not obliged to install.
- (6) We will cover the expenses necessary in connection with examination and subsequent performance, in particular as regards transport, travel, labour and materials if a defect does indeed exist. In the event of the unauthorised application of rights from defects, the buyer is obligated to compensate all the costs incurred (in particular costs for testing and transportation), unless the defect was not identifiable for the customer.
- (7) If the subsequent solution fails or a reasonable deadline which is to be set by the customer for the subsequent solution expires unsuccessfully or

it is dispensable according to the statutory regulations the customer can cancel the purchase contract or reduce the purchase price. This right of withdrawal does not exist with an insignificant defect.

(8) Claims of the customer for damages or reimbursement of fruitless expenses shall only exist according to number 9 of these GTC and are incidentally excluded, even if defects are present.

## 9. Other liability

- (1) Unless nothing else results from these GTC including subsequent clauses, we will be liable for violation of the contractual and non-contractual duties according to the legal regulations.
- (2) We shall be liable for compensation - on any legal ground - in the scope of fault-based liability in case of intent or gross negligence. In cases of minor negligence, we are only liable within a milder standard of liability in accordance with the statutory regulations (e.g. for the care in our own affairs) for:
  - (a) damages resulting from the destruction of life, personal injury or health damages
  - (b) damages from the breach of an essential contractual duty (obligation, the satisfaction of which only enables the proper execution of the contract at all and with which the contractual partner relies and may as a rule rely on its compliance); in this case our liability is however limited to the reimbursement of the foreseeable, typically occurring damages.
- (3) The limitations of liability stated in No. 9.2 of these General Terms and Conditions do also apply for breach of duty through or in favour of persons, which faults according to the statutory regulations we are entitled to be liable for. They do not apply in those cases in which we have maliciously concealed a defect or have furnished a guarantee for the condition of the goods as well as for claims of the customer in accordance with the law on liability for defective products.

## 10. Limitation

- (1) Notwithstanding § 438 Par. 1 No. 3 of the German Civil Code, the general statute-of-limitations for claims from defects of quality and title is one year from delivery. If acceptance has been agreed, the limitation period begins with the acceptance.
- (2) The above-mentioned limitation period shall also apply to the consumer's contractual and extra contractual compensation claims based on defective goods, except if the use of the standard legal limitation period (§§ 195, 199 of the German Civil Code) results in a shorter limitation period in individual cases. Apart from that, the statutory limitation periods according to number 9.2.1 and 2 of these GTC as well as according to the German Product Liability Act apply exclusively for damage claims of the buyer.

## 11. Choice of law, place of performance, jurisdiction

- (1) The subject General Terms and Conditions and all legal relations between us and the customer shall be governed by the law of the Federal Republic of Germany under exclusion of international uniform law, in particular of the UN Sales Convention. The place of performance for all services made by the customer is Hamburg.
- (2) If the customer is a merchant in terms of the German Commercial Code or if the customer is a public corporation or public asset, our registered office in Hamburg is the exclusive - and also international - place of jurisdiction for all disputes arising from this contractual relationship. This shall also apply if the customer is an entrepreneur within the meaning of § 14 of the German Civil Code. However, we are also entitled to take action at the general legal venue of the customer. Statutory regulations of prime importance, in particular exclusive jurisdictions, remain unaffected.

Hamburg, April 2016

**steco-system-technik GmbH & Co. KG**

Kollastr. 6  
22529 Hamburg  
Germany

T.: +49 (0)40 55 77 81-0  
F.: +49 (0)40 55 77 81-99  
E-Mail: [info@steco.de](mailto:info@steco.de)

[www.steco.de](http://www.steco.de)